

Clinical Trial: MicroPulse™ Nd:YAG Photorejuvenation with Skin Tightening



Pre-Treatment Detail

**Aletha Tippet MD
Cincinnati, OH**

INDICATION: Mild wrinkles and skin laxity along jawline, upper chin and corners of mouth.

ANESTHESIA : None.

TECHNOLOGY: MicroPulse™
1064nm Nd:YAG laser
(FriendlyLight, Tarrytown, NY).

SETTINGS: Energy Mode 4
(14 J/cm²), collimated beam,
6mm spot size, 1 Hz Repetition Rate

TREATMENT: single pass across
targeted areas, single treatment

Clinical Trial: MicroPulse™ Nd:YAG Photorejuvenation with Skin Tightening



**Detail, 10 days post
single treatment**

**Aletha Tippet MD
Cincinnati, OH**

INDICATION: Mild wrinkles and skin laxity along jawline, upper chin and corners of mouth.

ANESTHESIA : None.

TECHNOLOGY: MicroPulse™
1064nm Nd:YAG laser
(FriendlyLight, Tarrytown, NY).

SETTINGS: Energy Mode 4
(14 J/cm²), collimated beam,
6mm spot size, 1 Hz Repetition Rate

TREATMENT: single pass across
targeted areas, single treatment